## Peristeen® Plus Transanal Irrigation System Cleaning and drying instructions

This is a tool to help you understand how to properly clean and store your Peristeen Plus Transanal Irrigation System. This should be performed every time after you have completed a Peristeen Plus irrigation.

### 1. Dispose of the catheter

Unlock the balloon catheter and dispose of it with normal household waste. Do not flush the catheter down the toilet.

### 2. Empty and dry the system

- Unlock the connector from the lid of the water bag.
- Flip the lid to open the water bag and pour excess water out.
- Keep the lid open and leave the water bag upside down to dry for at least 1 hour.
- Drain the rest of the system by placing the control unit with connected tubes over the sink and turn the knob to the **finish and storage symbol** 
- Hang the control unit, including tubes, upside down and leave to dry for at least 1 hour.

All components should be stored at room temperature and out of direct sunlight. Ensure the tubes are not kinked and the system and all its parts are kept away from sharp objects.

### 3. Clean the system

**Apart from the single-use balloon catheter, all components can be washed in mild soapy water when needed. The strap can be washed in warm water (140°F) and hung to dry. The tube with the turquoise connector can be replaced if it becomes soiled.**

It is not necessary to disassemble the system before cleaning the external surfaces

**The internal surface of the components (including the tubes and control unit) can be cleaned by the following these steps:**

- Unlock the connector from the catheter
- Flip the lid to open the water bag and fill it to the top with mild soapy water.
- Close the lid by clicking it into place.
- Turn the knob on the control unit to the turquoise water symbol  and pump the water through the tubing into a sink until the water bag is empty.

Repeat this procedure twice with clean tap water to remove all of the soap. Then, unlock the connector from the lid and follow the drying procedure (as described above).

 Prior to use, refer to the product 'Instructions for Use' for intended use and relevant safety information.

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[www.coloplast.us](http://www.coloplast.us) El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S. © 2021 Coloplast Corp. Todos los derechos reservados.
Coloplast Corp. Minneapolis, MN 55411 / 1-800-533-0464