# Step-by-Step Guide (Adult)

Watch full how-to-use video here: <http://coloplast.to/peristeenplus-howtouse-adult>

## A. Getting ready


Empty the bladder before irrigation.

1. Flip the lid to open the water bag. Fill the water bag to the top with clean lukewarm tap water or bottled water. The round temperature indicator on the water bag will be green when it is within a suitable range to begin irrigation (step 10).

2. Attach the tube with the gray connector to the gray screw top by aligning the big tube with the big hole and the small tube with the small hole. Make a "T", and then twist the connector clockwise to lock.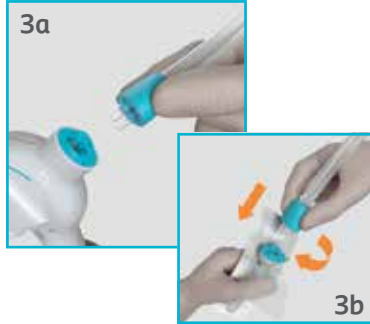

3a. Attach one end of the tube with turquoise connector to the control unit by aligning the big tube with the big hole and the small tube with the small hole. Make a "T", and then twist the connector clockwise to lock, until you feel a subtle click. 3b. Do the same to attach the other end of the tube with turquoise connector to the balloon catheter.

4. Attach the catheter to a wall by using the adhesive dots or place it, upright in its packaging, in a cup or a glass.

5. Turn the knob to the turquoise water symbol. Slowly pump water into the catheter packaging. Leave the catheter in the water for at least 30 seconds to lubricate it.

Consider doing steps 1-5 in the sink to contain any water spill.

## A. Getting ready

6. Turn the knob to the white balloon symbol to stop the water. Remove the lubricated catheter from the packaging and use it within 2 minutes.

## B. Inserting catheter

7. Sitting on the toilet or commode chair: Hold the catheter by the finger grip, just below the turquoise dot. Carefully insert the balloon catheter into the rectum as instructed by your healthcare professional, leaving the finger grip outside of your body.

## C. Inflating the balloon

8. When the catheter is inserted, pump to inflate the balloon. Recommended amount of pumps by your HCP: \_\_\_\_\_

(Maximum 4 pumps for the regular catheter, or 2 for the small catheter.)

9. Gently pull the catheter back to seal off the rectum.

## D. Pumping water

10. Turn the knob counter-clockwise to the turquoise water symbol following the circular line. Slowly pump in the water. Volume of water recommended by your HCP: \_\_\_\_\_ ml.

Do not use force when inserting the catheter. If you feel any resistance or obstruction, stop and contact your healthcare professional.

If you sense that the balloon is too big, turn the knob to the gray deflate symbol to deflate it.

Be careful not to turn past the gray deflate symbol, as this will deflate the balloon.

## E. Emptying bowel

11. Once the recommended volume of water is instilled, turn the knob counter-clockwise to the gray deflate symbol to deflate the balloon. The catheter may then fall out on its own. If not, gently remove the catheter from your rectum.

Sometimes water and feces come out right away, other times it can take longer.

12. Twist the catheter off. Dispose of it with household waste.

13. Stay seated on the toilet or commode chair until you stop passing water and stool.

Try massaging your abdomen or leaning forward to help your bowel do its job. The time this takes varies by individual, but on average you can expect to wait about 30 minutes. Some users treat themselves to an episode of their favorite TV show while they wait on the toilet.

## F. Cleaning up

14. Turn the knob to the finish and storage symbol. Empty the water from the water bag and tubes. Let the equipment dry first before storing it in a dry place away from direct sunlight. Finish up by washing your hands.

Turning the knob to the finish and storage symbol releases the remaining water from the tubes.

### Control unit symbols

Turquoise water symbol  
To pump water

White balloon symbol  
To inflate the balloon on the catheter and stop the water

Gray deflate symbol  
To deflate the balloon on the catheter

Finish and storage symbol  
To dry the device after use and when storing

It's very important to develop and maintain a good routine with the frequency of use prescribed to you by your healthcare professional.

If you need some product and lifestyle support, don't hesitate to reach out to our trained Coloplast® Care Advisors.

1-855-605-7594  
[peristeen@coloplast.com](mailto:peristeen@coloplast.com)

Before use, always consult the Instructions-For-Use (IFU) document delivered with the products.

**Indication:** The Peristeen® Plus Transanal Irrigation is intended to instill water into the colon through a rectal catheter, which incorporates an inflatable balloon, inserted into the rectum to promote evacuation of the contents of the lower colon. The Peristeen Plus Transanal Irrigation System is indicated for use by children (2 years - <12 years old), adolescent (12 years - < 18 years old), transitional adolescent (18 - <21 years old) and adult patients with neurogenic bowel dysfunction who suffer from fecal incontinence, chronic constipation, and/or time-consuming bowel management procedures.

**Contraindications:** Peristeen Plus Transanal Irrigation must not be used in the following situations: known anal or colorectal stenosis, colorectal cancer (active/recurrent), radiotherapy to the pelvis, and recent abdomino-perineal surgery, active inflammatory bowel disease, acute diverticulitis, severe diverticulitis, previous diverticulitis and diverticular abscess, chronic symptomatic diverticular disease, within 3 months of abdominal, anal or colorectal surgery, within 4 weeks of endoscopic polypectomy, recent colonic biopsy, recent endoscopic mucosal resection (EMR) and recent endoscopic sub-mucosal dissection (ESD), severe autonomic dysreflexia, ischemic colitis, during Spinal Cord Shock Phase, complex diverticular disease, in patients who are pregnant and have not used the system before\*\*. Since the list is not exhaustive, the healthcare professional should always consider individual patient factors as well.

\*\*If the patient is pregnant and has never used transanal irrigation before, they should not start the irrigation procedure during pregnancy.

**Warning:** Peristeen Plus Transanal Irrigation procedure should always be carried out with caution. Bowel perforation is an extremely rare, but serious and potentially lethal complication to transanal irrigation and will require immediate admission to a hospital, often requiring surgery. Seek immediate medical assistance if during or after Peristeen Plus Transanal Irrigation procedure you experience any of the following: severe or sustained abdominal pain or back pain, especially if combined with fever; severe or sustained anal bleeding; blood in the feces; rectal pain, and abdominal pain. See the device manual for complete user instructions, contraindications, warnings, cautions. For further information, call Coloplast Corp at 1-855-605-7594 or consult the company website at [www.coloplast.us](http://www.coloplast.us). **Caution: Federal law restricts this device to sale by or on the order of a physician.**

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